

CVAD Securement devices are used to stabilize and prevent CVAD dislodgement. At SickKids, we are changing products and this requires a practice change for CVAD dressing procedures.

Stop

The new CVAD securement devices are called Klik-Fix® by B Braun. There are **two sizes** to replace 3 StatLock™ products.

Small Size
For CVADs
3Fr or less



Large Size
For CVADs
4Fr or less



Think

How are CVAD dressing change procedures **changing** with Klik-Fix®?


1. Remove old CVAD dressings with care to avoid inadvertently opening the securement straps
2. Use ANTT to clean the areas of the CVAD that will be under the dressing – this maybe further than previously done
3. Add the Klik-Fix® to the critical aseptic field when it needs to be applied before the main dressing
4. Ensure that the Klik-Fix® is covered entirely by the main dressing and/or additional dressing tapes

Watch the Klik-Fix Intro Video for more details

Act

Secure a CVAD in **4 easy steps** using Klik-Fix®

- ☑ Expose adhesive patch on front of Klik-Fix® and adhere to CVAD wings
- ☑ Form a chevron with the securement straps and press into hook and loop material to secure
- ☑ Remove release paper backing one side at a time to place the Klik-Fix®
- ☑ Apply main dressing and/or additional dressing tapes to cover the Klik-Fix®

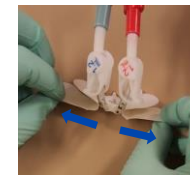

Review

Introducing new CVAD securement devices is a **practice change** and requires us to maintain a questioning attitude.

- **Stop.Think.Act.Review.** Before completing a CVAD dressing change with Klik-Fix®
- Reach out to Connected Care Live for educational support and coaching
- Review with patients and caregivers that a new securement device is being used
- Perform CVAD dressing and site assessments as per policy
- Report any CVAD or securement device issues through your organizational safety reporting system